


TROD'ENGINE® Strip

Procedure*

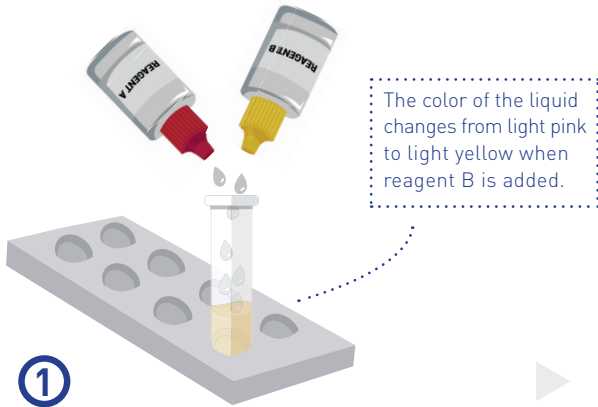

①

Dispense **4 drops** of reagent **A** then **4 drops** of reagent **B** in the extraction tube



②


Take the **sample** from the throat using a swab (avoid teeth, gums, tongue and cheeks)



③

Introduce the swab into the extraction tube then perform **10 rotations**

Wait a minute

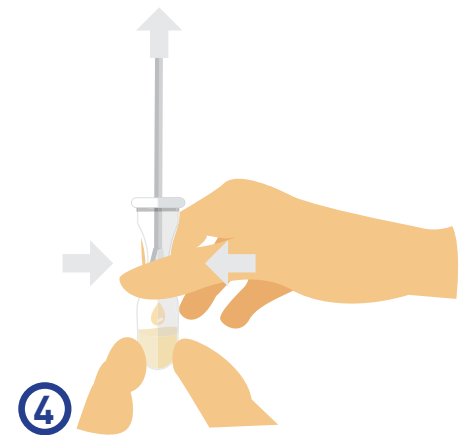

④

Press the **swab** against the sides of the tube and discard it



⑤

Immerse the end of the strip, arrows pointing down into the extraction tube

Read the result at **5 minutes**



Do not read after 10 minutes

INTERPRETATION

Positive result



Negative result



Invalid result



*Read the entire procedure before carrying out the first test.

Update : February 2020